NaturDent[®]

Liquid Cleanser

Daily Cleansing

Remove your dentures and brush them under running water with a toothbrush.

Drop half or one tablet into warm water. If possible leave the denture in the solution overnight.

Rinse denture under running water.

Dry denture thoroughly before applying **NaturDent** Denture Adhesive.

Cleansing once a week with NaturDent Liquid Cleanser

NaturDent Denture Adhesive contains a patented formula of coniferous resins. Use the NaturDent natural cleansing liquid once a week in order to get rid of the residues of the coniferous resins in a natural way. After cleaning the denture with a cleansing tablet, spray the NaturDent liquid on the dry denture and wait a few minutes. Then spray the liquid on a soft cloth and rub the denture with it.

Your denture will be fresh and clean!

Composition: Alcohol, Ricinus Communis Oil (Castor Oil), Melaleuca Alternifolia Oil (Tea Tree Oil).

NaturDent®


Denture Adhesive

Directions for Use:

1. Clean and dry your dentures thoroughly. **NaturDent** can only develop its full adhesive power when the dentures are completely dry.

2. Apply **NaturDent** to your dentures in strips (as shown in the diagram), making sure that the adhesive is not too close to the edge of the denture.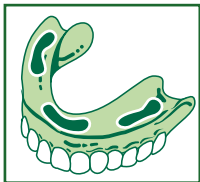

3. To avoid clogging, close tube tightly immediately after use.

4. Insert dentures into mouth and bite firmly.

5. For best result wait 10 min. before eating or drinking.

Helpful Hints:

- It may take a few trials to see what amount of **NaturDent** adhesive is ideal. First begin by using a small amount. Increase as needed to achieve a long lasting bond.
- If you wish an even stronger bond, spread adhesive to create a thin and even layer over the entire contact area.
- If at the end of the day the bonding effect has not lessened, simply leave dentures in the mouth overnight. Apply less adhesive next day.
- Consult your dentist regularly to ensure proper fitting dentures.
- Close tube tightly immediately after use.

Composition: Sodium Carboxymethylcellulose, Solution of Polyvinylacetate, Ester of Wood Rosin, Coconut Oil, Silica.

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Manufacturer: Fittydent International GmbH, A-7423 Pinkafeld, Austria
Hersteller: Fittydent International GmbH, A-7423 Pinkafeld, Austria