

# **fittydent**<sup>®</sup> *The waterproof Denture Care System*

Congratulations for choosing zinc free\* **fittydent sensitive**. It is the patented, waterproof adhesive that combats the problems of gum irritations from ill-fitting dentures. **fittydent sensitive** with soothing Aloe Vera and Myrrh not only gives you a strong whole day holding power, it eliminates slipping and sliding and prevents dentures from rubbing the gum.

Because the adhesive is waterproof, **fittydent sensitive** won't wash away while eating and drinking. It gives you long-lasting hold for up for a whole day. And you won't swallow any dissolving particles.

Thanks to its extraordinary hold, **fittydent sensitive** with Aloe and Myrrh also helps significantly reduce gum problems, such as irritations and pressure points caused by dentures rubbing.

**fittydent sensitive** gives you the confidence to eat, drink and smile and not worry about your dentures or about adhesive particles leaking into your stomach.

With **fittydent sensitive** you can count on the proven quality of the **fittydent** brand, which stands for long and strong adhesion of dentures which help prevent irritations of the gums.



\* Most of the denture creams contain zinc. Chronic, excessive intake of zinc may result in serious neurological disease. This is the conclusion of the study "Denture cream: an unusual source of excess zinc, leading to neurological disease" published in Neurology 2008.

Manufacturer: Fittydent International GmbH, A-7423 Pinkafeld, Austria  
[www.fittydent.com](http://www.fittydent.com)

# fitty dent<sup>®</sup>

sensitive

## Denture Adhesive with Aloe Vera & Myrrh

### Directions for Use:



1. Clean and dry your dentures thoroughly. **fittydent** can only develop its full adhesive power when the dentures are completely dry.
2. Apply **fittydent** to your dentures by squeezing the tube slightly. Warm the tube in hand if it is cold to let the adhesive come out easily.
3. Apply **fittydent** to your dentures in strips (as shown in the diagram), making sure that the adhesive is not too close to the edge of the denture.
4. To avoid clogging, close tube tightly immediately after use.
5. Insert dentures into mouth and bite firmly.
6. For best result wait 15 min. before eating or drinking.

### Helpful Hints:

- It may take a few trials to see what amount of **fittydent** adhesive is ideal. First begin by using a small amount. Increase as needed to achieve a long lasting bond.
- If you wish an even stronger bond, spread adhesive to create a thin and even layer over the entire contact area.
- If at the end of the day the bonding effect has not lessened, simply leave dentures in the mouth overnight. Apply less adhesive next day.
- For sensitive gums, a mild tingling or burning sensation may be felt occasionally. It will pass by quickly and is absolutely harmless. To avoid the tingling or burning sensation spread the adhesive to create a thin and even layer. Wait a few seconds before inserting the denture.
- Use **fittydent** Cleansing Tablets regularly to thoroughly clean dentures.
- Soft liners might reduce bonding effect.
- Consult your dentist regularly to ensure proper fitting dentures.

**Composition:** Sodium Carboxymethylcellulose, Solution of Polyvinylacetate, Triacetin, Paraffinum Liquidum, Silica, Aloe Vera Gel, Commiphora Myrrha Extract.

**Store at room temperature!**



**One Application Lasts All Day. Apply Less Often, Save Money!**



# Denture Adhesive Cushions


## for lower dentures

### Directions for use:



1. Clean and **dry** your lower denture thoroughly before applying **fittydent** Adhesive Cushion.

2. Remove the foil on both sides of cushion before using. To easily remove the foil, bend cushion slightly.

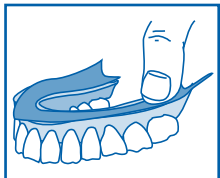

3. Apply the cushion inside the lower denture and press firmly in place. Gently smooth out any air bubbles.



4. If necessary, trim cushion with scissors to fit denture.

5. Insert denture into mouth and bite firmly. For best results, wait 15 minutes before eating or drinking.

### Helpful Hints:

- For sensitive gums a mild tingling or burning sensation may be felt occasionally.
- Use **fittydent** Cleansing Tablets regularly to thoroughly clean dentures.
- **fittydent** Adhesive Cushions are not intended to use with soft lining dentures.
- Consult your dentist regularly to ensure proper fitting dentures.

Ingredients: Polyvinylacetate, Sodium Carboxymethylcellulose, Glycerintriacetate, Petrolatum.

**Store at room temperature!**



**The Only Guaranteed Solution For Lower Jaw Dentures.**

# fittydent<sup>®</sup> super


## Cleansing Tablets



### Direction for use:



Remove your dentures and brush them under running water with a toothbrush.

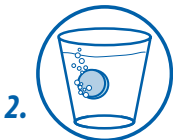

Drop half or one tablet into warm water.



Soak denture or brace in the solution for at least 5 minutes. For best results, use once a day. Your dentures or braces may be safely soaked in the solution overnight.



Rinse denture or brace under running water. Dry denture thoroughly before applying **fittydent** Denture Adhesive or Cushions.

**Caution: Keep tablets out of reach of children. Do not put tablets in mouth.**

Ingredients: Sodium Perborate, Sodium Bicarbonate, Potassium Monopersulphate, Trisodium Phosphate, PEG-240, Sulphamic Acid, PVP, TAED, Silica, Sodium Methyl Oleoyl

Taurate, Cellulose-Lactose, Colour C.I. 42090, Aroma

**Store at room temperature!**



**Freshen & Clean Dentures and Orthodontic Braces**

**with fittydent's Powerful Anti-Bacterial Formula**